VULPES GOODS
HEALTHCARE

MESH NEBULIZER

MANUAL • ANLEITUG
HANDLEIDING • MANUEL

WWW.VULPESGOODS.COM

ENGLISH	3
The Device	4
User Guide	7
Characteristics and Application	9
Problem and solution	11
Product parameters	12
DEUTSCH	13
Das Gerät	14
Benutzerhandbuc	17
Eigenschaften und Anwendung	19
Problem und Lösung	21
Produkt parameter	22
NEDERLANDS	23
Het apparaat	24
Gebruiksaanwijzing	27
Kenmerken en Toepassingen	29
Probleem en oplossing	31
Product Parameters	32
FRANCAIS	33
Le dispositif	34
Manuel de l'utilisateur	37
Caractéristiques et applications	39
Problème et solution	41
Paramètres du produit	42

ENGLISH

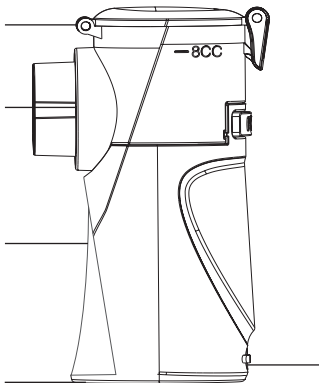
THE DEVICE

Medicine cup


Atomizer

On/Off
button


Battery
compartment


Adaptor
connector

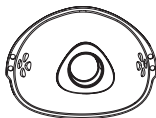Adults
Mask

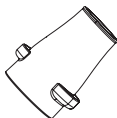Mouthpiece

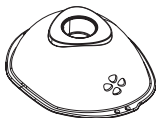Children
masks

USB cable

Medicine cup

Fill the medicine cup with the desired and prescribed liquid and make sure the medicine cup is correctly connected. If the medicine cup is not connected correctly, the nebulizer will not work.

Atomizer

Always keep the atomizer free of residue and ensure proper cleaning after use. This promotes the life of the atomizer and can ensure better results.

On/Off button

Press once to turn on the nebulizer.

Press twice to increase the speed of nebulization.

Press three times to turn off the nebulizer.

Battery compartment

Open the compartment to insert the batteries.

Observe the correct connection for + and -. Without the correct connection, the nebulizer will not work.

Adaptor connector

Attach the adapter to the back if you choose to work with the adapter. Make sure the connection remains good. Without a constant connection, the nebulizer will not work.

Adult mask

Choose the appropriate mouthpiece based on the desired connection. The adult mask is the largest version.

Mouthpiece

Choose the right nozzle based on the desired connection.

Children's mask

Choose the appropriate mouthpiece based on the desired connection. The children's mask is the smallest version.

USB cable

Use the supplied cable if you do not have batteries available. The nebulizer must have constant supply of power to work correctly.

USER GUIDE

1. Choose the desired medication or liquid and fill the medicine cup with it. You can easily open the medicine cup by lifting the clip on the back. Never exceed the recommended amount.

2. After filling, snap the medicine cup onto the nebulizer until you hear a “click.” If the medicine cup is not attached correctly, the nebulizer will not work.

If you wish to remove the medicine cup, you can press the “push” button and gently remove the medicine cup.

3. Choose the correct power supply. The nebulizer should be constantly supplied with power. (2x AA batteries or DC5V adapter).

4. Press once to turn on the nebulizer.
Press twice to increase the speed of nebulization.
Press three times to turn off the nebulizer.

If a component is not connected correctly, the upper light will illuminate briefly, then the nebulizer will turn itself off.

5. After use, remove the nozzle or other accessories from the nebulizer. Follow the instructions below for proper cleaning.

6. Add 6 milliliters of clean water to the medicine cup and allow it to atomize completely until the medicine cup is empty. This ensures that all residue goes out of the nebulizer and this prevents clogging.

7. Remove the batteries or adapter and store them safely. Removing the batteries is critical to the durability of the product.

8. Make sure the nebulizer is dry when you store it. Remaining residue may solidify and cause blockages.

Cleaning the nebulizer is crucial for optimal performance and preventing further infections.

CHARACTERISTICS AND APPLICATION

- Do not put pressure on the atomizer. This may actually damage the mesh and will not be covered under warranty. The metal is very thin and can be damaged quickly.
- Before cleaning the device, always remove the batteries or adapter.
- Do not store the device in a damp room (e.g. bathroom) or transport it together with damp objects.
- Store the device away from sunlight.
- Proper breathing technique is important to ensure that the particles of the medication are widely distributed through the airways. To ensure that this happens, take deep, slow breaths for about 5 to 10 seconds and then exhale quickly.
- Use the nebulizer only to treat respiratory conditions on the advice of your physician.
- Some medications are available only with a prescription.
- When replacing batteries, make sure the medicine cup is completely empty, otherwise there is a risk of leakage.
- When the red LED light comes on, replace the batteries, atomizing becomes less effective when the batteries are dead.

- If you are not going to use the device for an extended period of time, remove the batteries from the device.
- Use only the prescribed medication or water. Do not use oils or thickened medication.
- For medical advice, we always recommend contacting a primary care physician or qualified professional. This may promote use.
- Do not leave children under the age of 16 alone with the nebulizer. Always assist with use for optimal operation.
- Remember that cleaning is crucial for optimal operation. Follow the instructions to avoid breakdowns.
- Do not interchange mouthpieces. This may worsen operation and spread bacteria.
- On position 1 (1x on/off button) the nebulizer will go into standby mode after 5 minutes. On position 2 (2x on/off button) the nebulizer will go into standby mode after 10 minutes. After this you can turn the nebulizer on again.

PROBLEM AND SOLUTION

The nebuliser produces little or no nebulisation	<ol style="list-style-type: none">1. Possibly there is too little medication in the nebuliser.2. The nebuliser is not held upright.3. The medication is not appropriate.4. The nebuliser has not been cleaned and contains dirt
The output is low	<ol style="list-style-type: none">1. The batteries are dead, replace them.2. There is air in the medicine cup and the medication is not flowing.3. The atomiser is obstructed, fill the medicine cup with 3 drops of vinegar and 5ml of water and let it atomise completely. Do not inhale this.4. The atomiser is worn out.
What medications are suitable?	Only a doctor or medical professional can advise which medication is suitable for you. Do not use medication with a viscosity higher than 3
Some of the liquid remains in the nebuliser	This is common and is due to technical reasons. Clean the medicine cup after use to avoid blockages.
What steps should I take for use on children and babies?	With babies and children, the mask should cover around the nose for effective inhalation.
Inhaling with the mask takes longer	This is because of technical reasons. Less medication is inhaled through the mask per inhalation.

PRODUCT PARAMETERS

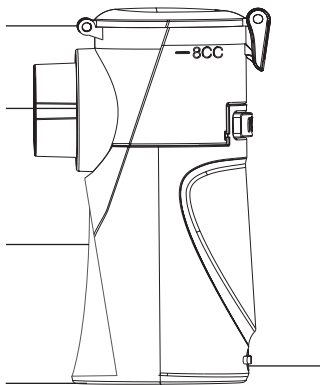
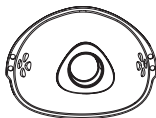
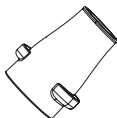
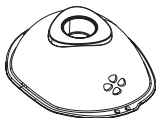
Power consumption:	2.0W
Input:	5V/1A;
Oscillation frequency:	110kHz +/- 10%
Condensation rate:	>0,2 ml/min.
Atomized particles:	< 5
Volume:	max. 8 ml
Product weight:	115 gr
Dimensions (lxwxh):	45 x 45 x 100mm
Housing material:	ABS
Operating conditions:	5-40 degrees Celsius
Temperature:	-20 - +50 degrees Celsius
Relative humidity:	<93%

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