**VULPES GOODS**  
BABYCARE

# DOPPLER

**PRO**

**MANUAL • HANDLEIDING**  
**ANLEITUG • MANUEL**

[WWW.VULPESGOODS.COM](http://WWW.VULPESGOODS.COM)

## **ENGLISH** **3**

The Device	4
User guide	8
Characteristics and Application	9
FAQ	11
Product Parameter	12

## **DEUTSCH** **13**

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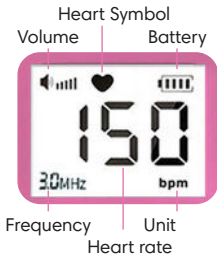
## LED Display

### Volume

Select the desired volume at which the heartbeat can be heard.

### Heart icon

The icon will flash as soon as heart rate is measured.

### Battery

Displays the current battery status.

### Frequency

Displays the current frequency being used.

### Heartbeat

Displays the measured heart rate. Note: If no heart rate is present, the screen will show - - - -. This can sometimes take 3 seconds.

## **Unit**

Shows the current unit being used.

## **Headphones**

Connect your earphones or headphones by plugging them in at the top.

## **Display**

The display provides all the information you need and is highlighted on the previous page. Choose an illuminated display by pressing the power button once once the unit is turned on.

## **Volume + / -**

Select the desired volume by pressing the up or down arrow. For the headphones, this works the same way.

## **On/Off**

Press the power button to turn the unit on. Press again to turn the display illumination on/off. Press and hold the button for 3 seconds to turn the device off.

## **Speaker**

From the speaker you can listen along with the heartbeat. The volume of the speaker is controlled by the volume knob. As soon as you connect earphones, the speaker will switch to the connected earphones.

## **Probe**

By placing the probe correctly, you can listen in on a heartbeat. **Note:** If no heartbeat is present, the screen will show - - - -. This can sometimes take 3 seconds.

## **Connecting cable**

The connecting cable is to connect to the probe. Make sure the connection is correct for proper operation.

## **Ultrasonic gel**

You can use the included gel while measuring. This ensures that the sound is better conducted.

# USER GUIDE

To use the doppler correctly, use the correct batteries. Also, always check that the probe is connected correctly.

Activate the doppler by pressing the on/off button. press. Then measure your own heart rate first to check the operation. Use the appropriate gel and then you can get started.

Never remove the connecting cable when the doppler is on. This may cause internal damage and cause the measurement to be incorrect.

**Note:**

If the doppler is not used for a minute, the doppler will turn itself off. After this, you can turn the doppler back on immediately.

# CHARACTERISTICS AND APPLICATION

- The doppler is not a toy. Keep it away from children. The recommended age is 16 years. Keep children at a safe distance from the device.
- The device operates on batteries. Do not allow water or moisture to reach the batteries. If you are not using the doppler for an extended period of time, remove the batteries.
- The storage temperature of the product should not be lower than -10 or higher than + 45 degrees Celsius. A storage temperature that is too cold or too hot will shorten the life of the device.
- Make sure direct contact is made between the skin and the gel. This improves the effect.
- Turn off the doppler if you wish to change the probe. Never do this when the doppler is on.
- Turn off the device immediately if unusual noises occur during use or if the device suddenly becomes very hot.

- Use the device in a clean and dry environment. Do not allow the doppler to come into contact with water.
- Keep the probe clean. A dirty or defective probe can cause an incorrect reading.

## FAQ

### **Q. I don't hear any sound?**

**A.** Check that the batteries are full enough. A low battery may result in softer sound.

**A.** The probe is not connected properly, therefore the measurement is not correct and you cannot hear anything

### **Q. If I don't measure a heart, I still see a heartbeat**

**A.** It usually takes 3 seconds for the probe to measure heart rate. Once no heart rate is present you will see - - - - on the display.

### **Q. How do I know if I am measuring my heart?**

**A.** The display shows the heartbeat and the heart icon will start beating.

### **Q. The probe does not measure heart rate?**

**A.** Check that you are measuring correctly and the location of the heart is correct.

**A.** The probe may be damaged or dirty. As a result, the measurement is incorrect or nothing can be measured.

**A.** Check that the connection is correct from the connection cable.

# PRODUCT PARAMETERS

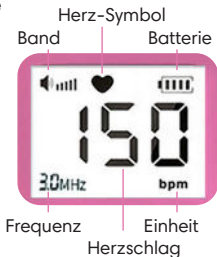
Model:	VG-T501
Weight:	190Gr
Dimension (LxWxH):	9.2cm x 3.4cm x13.5cm
Working time:	4 hour
Storage temperature:	-10 - + 45 °C

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